

## Extra Efficient !


### Radius Quad set up alignment

The Radius Quad set up procedures are a lot like the other Monotrac set ups except it is important to pay closer attention to trimming the impression flat and parallel to the occlusal plane and to pre align the tray over the impression and mark it with a Sharpie pen. These two tips will greatly help in achieving a nicely aligned set up.



Trim the impression flat and parallel with Monotrac Trimmers. Trace a Sharpie around the outside radius and also indicate the hash mark ribs of the tray onto the impression. When the impression and tray are poured, the tray is inverted over the impression using these marks to guide centered optimal alignment.

**Monotrac Trimmers** - used for trimming impressions flat and parallel for straighter cleaner set up. Stripping tear away walls and cutting excess stone flash from the models.

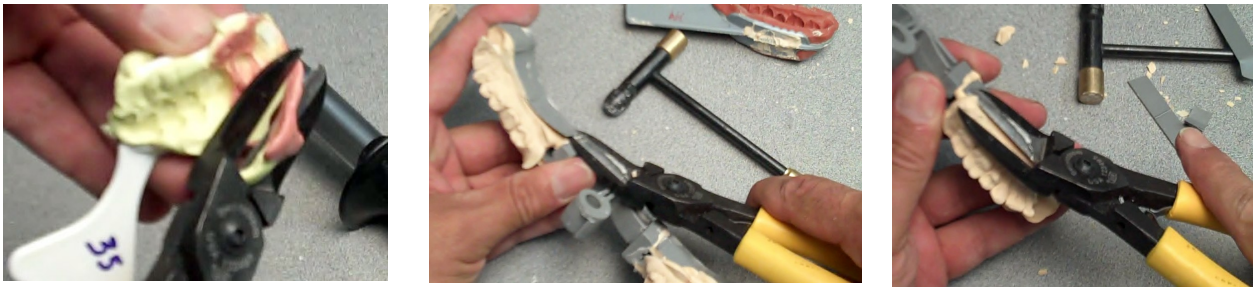


**Monotrac Speed Spatula** - Holds and pours more stone faster and with better placement control.



Checking clearance



Invert pour over the impression

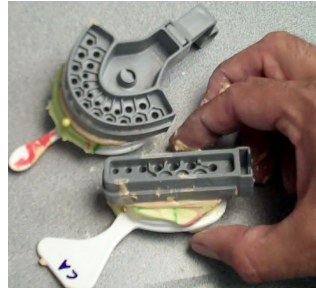

Scrape off tear tags & mark before you eject model

