

Monotrac Split-Cast Adapter Plate



The Split-Cast Adapter Plate is a magnetic plate that allows easy transfer of the Monotrac V2 Full-Arch models to Semi or Fully Adjustable articulators.

It allows easy transfer from the Flex-Arm set up to a large articulator if desired, First complete the model setup as usual onto the V2 Flex-Arm Articulator. The Adapter Plates are magnetically attached to the bottom side of the V2 Base. Attach the washer plate to the upright center post under the V2 Base with supplied attachment screw. Do not over-tighten the washer but allow it to rock slightly for closer magnetic contact. You can adjust the strength of the magnetic connection in this manner.

It is then plastered into position on the Semi or Fully adjustable articulator. (TIP) Paint a coat of vaseline on the back of the plate before plastering for easier clean-up for next reuse.



You can now remove the V2 Bases from the articulator Split-Cast plates. Remove the V2 hinge to allow the models to function on the adjustable articulator. The technician now can do the build-up or waxing procedures on the easy-to-handle Monotrac Articulator then, as needed, move to the Adjustable for any required mechanically guided refinements.

1. Remove and replace the Monotrac Flex Arms, and Plug-In Stops as you transfer between articulated setups.
2. Kits include 4 Split Cast Adapter Plates with magnets, 32 washer plates, and 32 attachment screws.

